

BED-MON

ULTIMATE IN
BED MAT
MONITORING



WHAT THIS SYSTEM CAN DO FOR YOU!!

Reliable—no fuss alternative to monitoring your residents movements in bed.

Staff Friendly— with ease of use (no setup) and less false alarms?

Flexibility—provide easy no fuss connection interface.

Totally Customisable—If you can dream it we can create it.

BFELECTRONICS

U4 2800 Logan Road, Underwood, Qld 4127
PO Box 1262, Springwood, Qld 4127
www.bfelectronics.com.au
bedmon@bfelectronics.com.au
Office: +61 7 3117 0552
Mobile: +61 478 068 088



**Pre-order
and reserve a
10%
Discount**

See next page for details

BED-MON

The new model of BED-MON is a MCU processor controlled "Smart" device. The processor is fully programmable (workshop), the programming has been written to counteract false alarms due to a person moving around in bed. Most bed mat interfaces just reverse the signal that needs to be applied to the Nurse Call system call point with no allowance for a person moving around in bed thus the possibility of false alarms.

This is where the BED-MON has been specifically engineered and programmed to monitor the initial movement of the bed mat and then waits for approximately 5 Seconds and if the signal indicates a legitimate alarm then and only then does it send an alarm to alert staff. The system will not reset until the person returns back to bed at which time it will reset itself and start monitoring again. Please note the BED-MON does not interface with the nurse call system operation thus normal operating procedures regarding the clearance of nurse call system alarms is still required.

All models of BED-MON can be customized to suit the connection requirements of your current Nurse Call system, irrespective of the system being Wireless or hardwired. A radio transmitter from well known manufacturers such as Smartlink, Advance Care, Smart-caller and more can be installed inside the BED-MON transforming the system into a portable monitoring system.

[There are 2 versions of BED-MON.](#)

VERSION 1. (Model ~ BED-MON)



This is the basic model and is not customisable other than the connection required by your current Nurse Call system. The basic version is fitted with one(1) input socket for the bed-mat and power jack.



VERSION 2. (Model ~ BED-MON P)

This is the BED_MON P and is fitted with the bed mat input jack and power jack but with an added input jack for the pendant cord, the pendant will still function even if you have turned the BED-MON off. The BED-MON P is not customisable other than the connection required by your current Nurse Call system.

Release Date Sept 2016 **Call Now** to secure yours on **0478 068 088** or online



Conditions Apply

Order before 30th September 2016 and receive all BED-MON's for the 5+ pricing (up to 10%).

Offer does not apply to accessories, externally purchased devices from other suppliers, installation or bed mats & floor mats.

All orders pertaining to this offer require COD or a \$30 deposit per unit paid with order and balance on a 14 day invoice.

Please Note: We are not registered for GST therefore all prices quoted are the final price.

Offer expires Friday September 30th 2016.

Model	Single item (1-3)	Quantity 5+
BED-MON	\$140.00	\$125.00
Add a Pendant socket	\$25.00	\$20.00
Bed mat Kit **	\$215.00	\$195.00
Bed Mat	\$75.00	\$70.00
Floor mat	\$215.00	\$205.00

BFE ELECTRONICS

** Kit includes 1 x BED-MON & 1 x Bed mat.
Delivery not included in prices above or current offer